


## USING THE BLUE2 as an IPAD CLICKER

When a student can press a switch, and you want to allow him/her repeated access to a specific button on an iPad app (e.g. the play button of Voice Dream Reader, or the page turner button of iBooks, or the photo shoot button of the Camera app, etc.) then this “iPad Clicker” option may be a solution for you.

### Three things to do to use the BLUE2 as an iPad Clicker:

- I. **PROGRAM THE BLUE2**
- II. **PAIR the iPad to the BLUE2**
- III. **Set up VOICEOVER**

#### I. PROGRAM THE BLUE2:

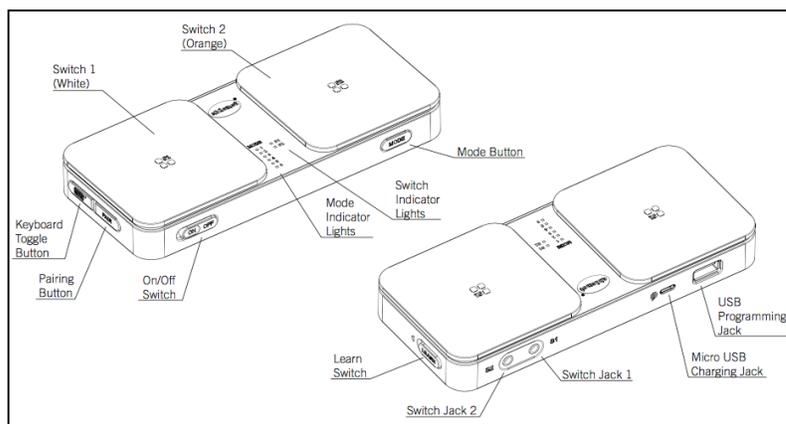

- a. Turn on the BLUE2 using the on/off switch.
- b. Plug a USB Keyboard into the USB programming jack.
- c. Turn on the Learn Switch by sliding it to the left.
- d. Press the Mode button repeatedly until the Mode Indicator Light shows that Mode 4 is selected.
- e. Press Switch 1 (White) to start the programming process. The S1 light will blink.
- f. On the USB keyboard, press simultaneously the keys CONTROL + OPTION + SPACEBAR.
- g. Press Switch 1 again to stop the programming process.
- h. Turn off the Learn Switch by sliding it to the right.
- i. Remove the USB Keyboard.

## II. PAIR the iPad:

- a. On the BLUE2:
  - i. Turn on the BLUE2 using the on/off switch (if it is not already on).
  - ii. Press the Pairing Button.
- b. On the iPad:
  - i. Tap on the SETTINGS app.
  - ii. On the left column of the Settings page, tap on BLUETOOTH to select it.
  - iii. On the Bluetooth page, switch on Bluetooth (if it is not already on).
  - iv. The BLUE2 device should soon appear on the page under Devices. Tap on it to connect.
  - v. Wait until the page indicates that the BLUE2 device is CONNECTED.

## III. Set up VOICEOVER:

- a. On the iPad Home Screen, tap on the SETTINGS app.
- b. On the left column of the Settings page, tap on GENERAL.
- c. On the General Page, tap on ACCESSIBILITY.
- d. On the Accessibility Page, scroll down to the bottom and tap on ACCESSIBILITY SHORTCUT.
- e. On the Accessibility Shortcut page, tap on VOICEOVER. This will allow you to eventually turn on Voiceover by clicking the Home Button 3 times in quick succession (triple-click).

---

## START USING THE BLUE2 as a CLICKER...

1. Make sure the BLUE2 is turned on and paired with the iPad (see instructions above).
2. On the iPad, open the app that you want to use, and navigate to the page that has the specific Button you want the BLUE2 to activate.
3. Triple-Click the HOME BUTTON of the iPad to turn on Voiceover.
4. Using 3 fingers, double tap on the iPad screen to turn off Voiceover's audio prompts.
5. Using 1 finger, tap once on the Button that you want the BLUE2 to activate. You should see a "rectangular border" around the Button that you have selected. This Button is now ready for BLUE2 to activate.
6. To activate the Button, click on Switch 1 (White) of the BLUE2.

**NOTE:** The BLUE2 needs Voiceover to be on in order to perform a "click" on the iPad. However while Voiceover is on, you will NOT be able to use the iPad screen in a regular manner. If you wish to turn off Voiceover, triple-click on the HOME BUTTON again.

**END** 😊