

DynavoxV Cheat Sheet

- Editing/Modifying a Button
 - Hit the GREEN round button on the top of the toolbar
 - The round button will then turn RED
 - Choose a button to modify by hitting the button
 - "MODIFY BUTTON" will pop-up
 - Change the LABEL (what you want the button to display) by hitting the text box next to where it says "Label"
 - A keyboard will pop-up. Type in what you want the label to be. Push OK when you are done.
 - To change the SYMBOL, hit the SYMBOL button. The symbol is the picture that is on the button.
 - "SELECT A SYMBOL" will pop-up
 - In the bottom right corner next to the SEARCH button there is a box. If the name of the picture you are searching for is already in the box, hit SEARCH.
 - Select the picture you want by hitting the picture
 - If you want to search for a different picture, hit the text box next to the search button
 - Type in the name of the picture you want to search then hit OK.
 - Choose the picture you want by hitting it.
 - Once you have selected your SYMBOL, hit the OK button at the bottom of the screen
 - If you want to modify the BEHAVIOR, hit the BEHAVIOR button.
 - Here is a list of what the behaviors will do:
 - **Clear**: will clear the message window
 - **Close Pop-Up**: closes the pop-up
 - **Go to Page**: Links one page to another
 - **Insert Label**: puts label in the message window (NOTE: When you hit the message window, whatever is in there will be spoken)
 - **Insert Text**: puts whatever text you type in in the message window (NOTE: You enter text whenever you want the button to say something other than the LABEL)
 - **Speak Label**: the device will speak whatever the label says
 - **Speak Text**: The device will speak whatever text you entered
 - The 2 columns you see are BEHAVIORS on the left and STEPS on the right
 - Whatever behaviors you want the button to do, you first need to highlight the behavior on the left side and then hit the ADD button toward the bottom of the screen. Do only one at a time. If you did it right, the behavior will show up under the left column (STEPS).
 - When finished, hit OK button on bottom of screen
 - The MODIFY BUTTON screen will return and hit OK if you are satisfied with your changes.
 - Test your newly modified button to be sure it does what you want it to.

If you have questions or need help on anything, call the Dynavox company for technical support. They are VERY helpful! The phone number is on the back of the device.

Remember this is a DYNAVOX 5.